



Product Registration



Register today to ensure your product is covered by warranty, and for easy access to product services and general support.

www.adeptmedical.com/form/product-registration



Warranty



Service




Support

Contents

Unboxing	4
Assembly	4
Product Use	5
Setup	7
Product Configurations	11
Spare Parts	12
Image Table Verification	13
Essential Information	14

ES Spanish	16	NO Norwegian	38
FR French	18	PL Polish	40
IT Italian	20	PT Portuguese	42
DE German	22	RO Romanian	44
NL Dutch	24	RU Russian	46
HR Croatian	26	SR Serbian	48
CS Czech	28	SK Slovak	50
DA Danish	30	SL Slovenian	52

Unboxing

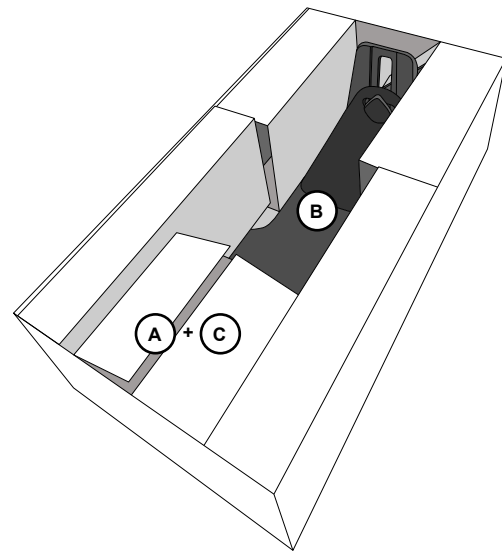

Head Immobiliser (AM0800)


(A) (B) (C)

	(A)	Hood Assembly x 1
	(B)	Base Board Assembly x 1
	(C)	Forehead Straps x 10


Hood Assembly + Forehead Straps (AM0820)

(A) (C)

Assembly

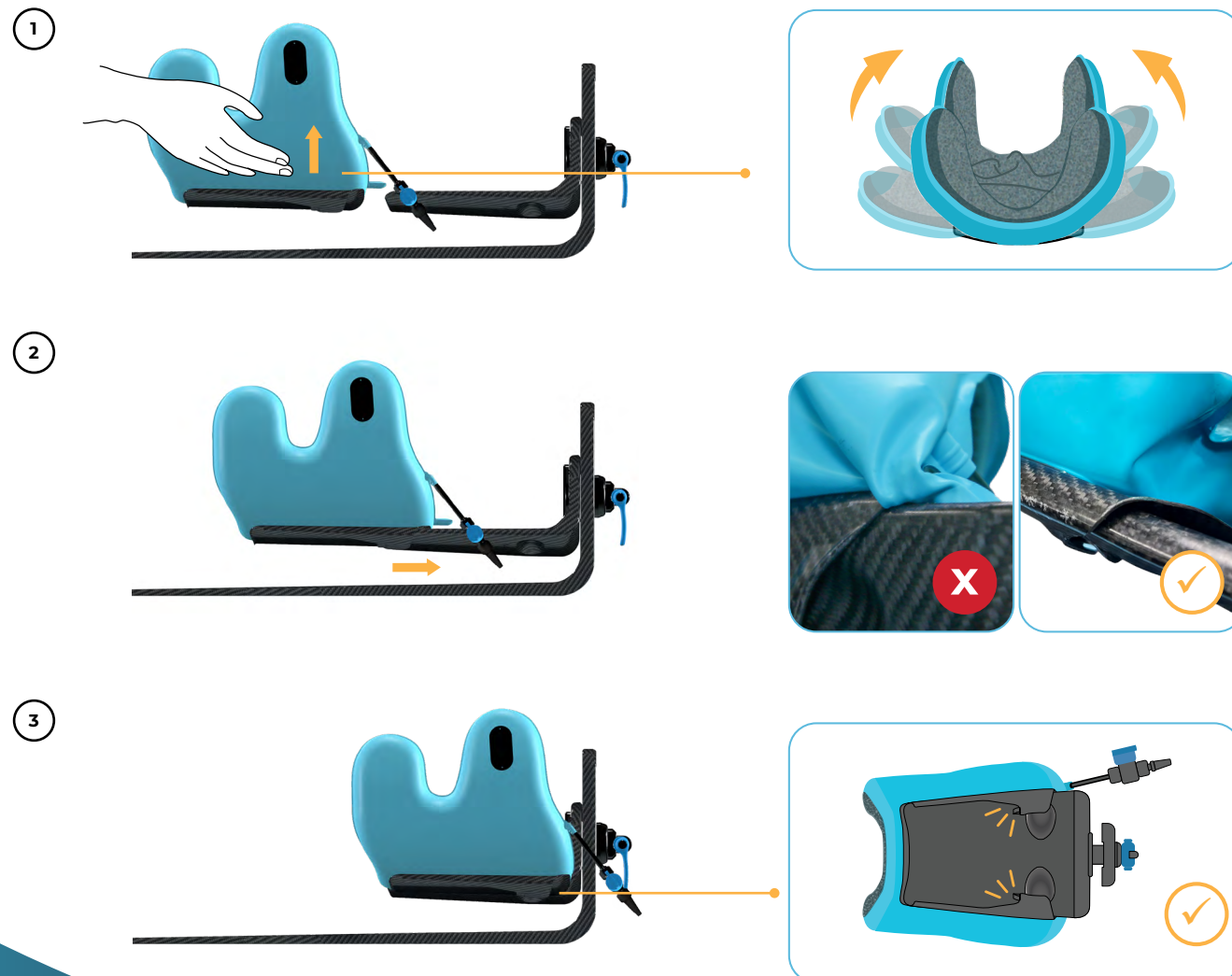


Hood Assembly Removal

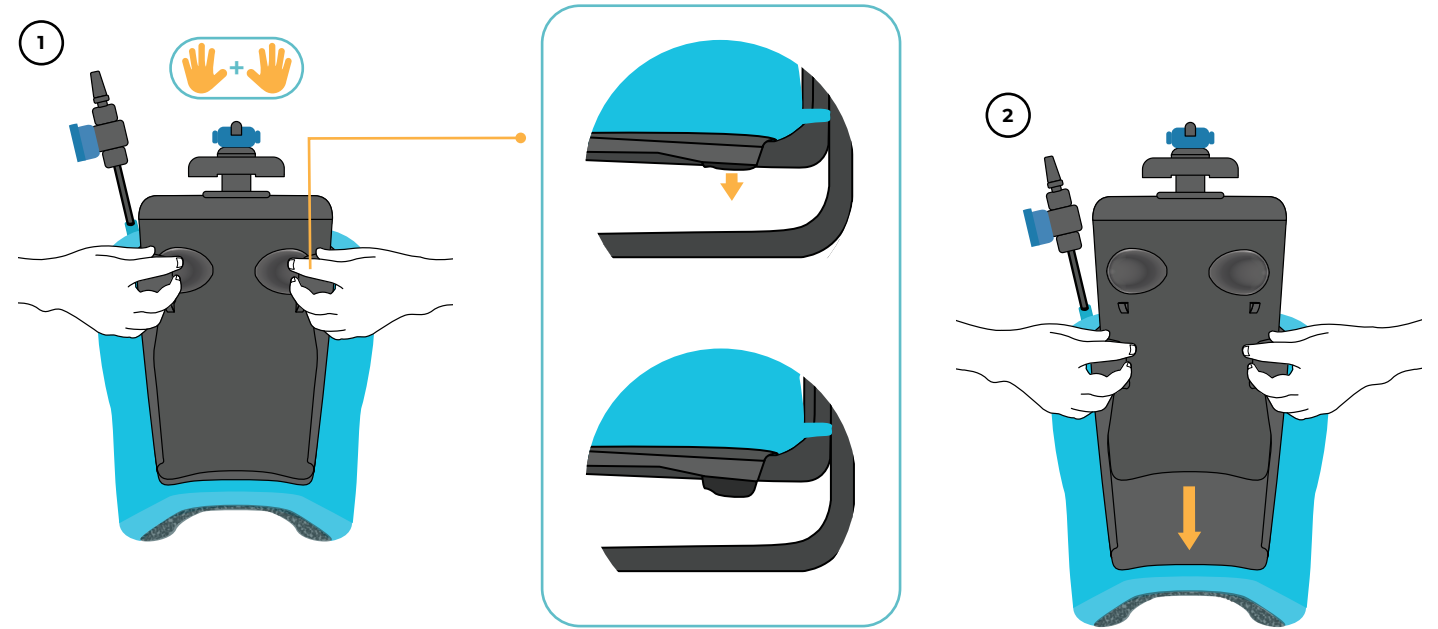

Product Use

Spare Parts

Product Codes

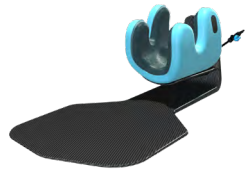
	Head Immobiliser	AM0800
	Hood Assembly + Forehead Straps x 10	AM0820

Component Codes

	Base Board Assembly x 1	M8000
	Base Board	M8100
	Hood Support	M8200
	Adjustable Assembly	M8300
	Forehead Straps x 10	AM0830

Hood Assembly Beads

If any beads appear outside the hood, DO NOT USE.

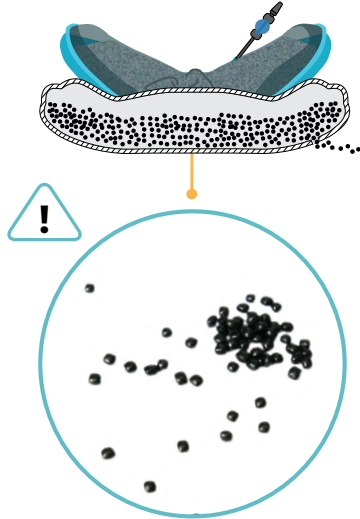

Essential Information

Information

Intended Purpose
To support, stabilise and position a patient's head during image-guided (CT and Angiography) medical procedures.

Intended User/Training Requirement
Intended to be used by trained medical professionals.

Patient Target Group
The intended patient is an Adult male or female requiring diagnostic imaging and image guided therapy.

- Contraindications**
- Not to be used with patients over 135 kg.
 - Not to be used for cases longer than 6 hours duration.
 - Not for paediatric use.
 - Not to be used on patients with Parietal thinning.
 - Not to be used on patient with focal calvaria thinning and other skull abnormality.
 - Not to be used on patient with severe cervical spine disease (osteoarthritis or rheumatoid arthritis or spondylarthritis).
 - Not to be used on patients that have had craniectomy or craniotomy performed in the last 2 years.
 - Not to be used on patient with recent surgery to the scalp, may affect wound healing.
 - Not to be used on patient with recent traumatic injury to the head.

- Warnings and Cautions**
- Ensure IFU is read prior to use.
 - Ensure Preparatory Cleaning is conducted.
 - Ensure Preparatory Product Check is conducted.
 - Do not use with patients over 135 kg.
 - Vacuum only, operating vacuum:
- | |
|----------------|
| Minimum |
| -50 kPa |
| -375 mmHG |
| -15 inHG |
- Check patient for pressure related injury after 2 hours.
 - 2 person set up.
 - Ensure head mass is supported if unlocking cam lever and adjusting position.
 - Not for paediatric use.
 - Remove hair clips, jewellery and eyewear.
 - Ensure long hair is flat against the back.

Incident Reporting Guidelines
For product complaints and incidents, please complete form on www.adeptmedical.co.nz/repairs



Medical
Supporting you



| adeptmedicaltraining.com



adeptmedical.com | 