RETROGRADE IR PLATFORM

AM0400 - AM0410 - AM0420

INSTRUCTIONS FOR USE

Issue Date: 03/2023

Revision: C

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Manufacturer
Adept Medical Ltd
2-6 McDonald St,
Morningside
Auckland 1025, New Zealand

ADEPT

Medical
Supporting you

 | adeptmedicaltraining.com

adeptmedical.com | 

Product Registration

Register today to ensure your product is covered by warranty, and for easy access to product services and general support.

www.adeptmedical.com/form/product-registration

Warranty

Service


Support

Contents

Unboxing	4
Assembly	4
Setup	5
Spare Parts	7
Image Table Verification	7
Essential Information	8

ES Spanish	10	NO Norwegian	32
FR French	12	PL Polish	34
IT Italian	14	PT Portuguese	36
DE German	16	RO Romanian	38
NL Dutch	18	RU Russian	40
HR Croatian	20	SR Serbian	42
CS Czech	22	SK Slovak	44
DA Danish	24	SL Slovenian	46
FI Finnish	26	SV Swedish	48
EL Greek	28	TR Turkish	50
HU Hungarian	30		

Unboxing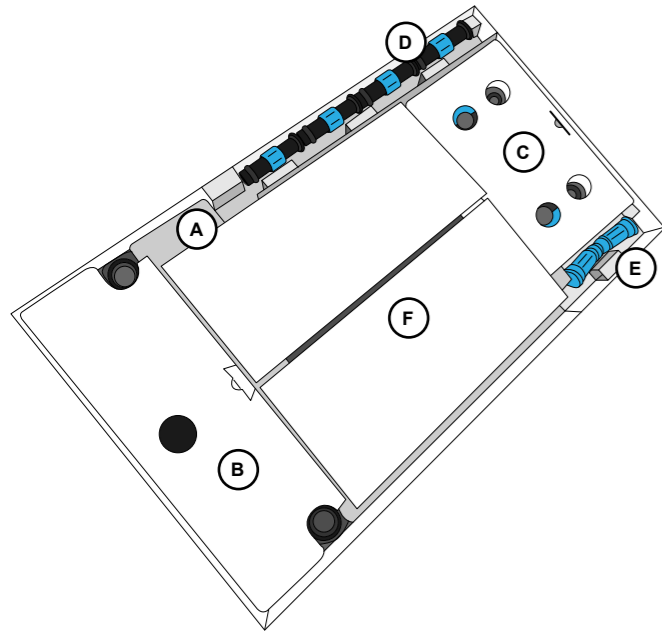

	(A)	Retrograde Main Platform x1
	(B)	Front Base x1
	(C)	Rear Base x1
	(D)	Leg x4
	(E)	Leg Extension x2
	(F)	Extension* x1



*AM0410 - AM0420

AM0400
(A) (B) (C) (D) (E)


AM0410
(F)





AM0420
(A) (B) (C) (D) (E) (F)

Assembly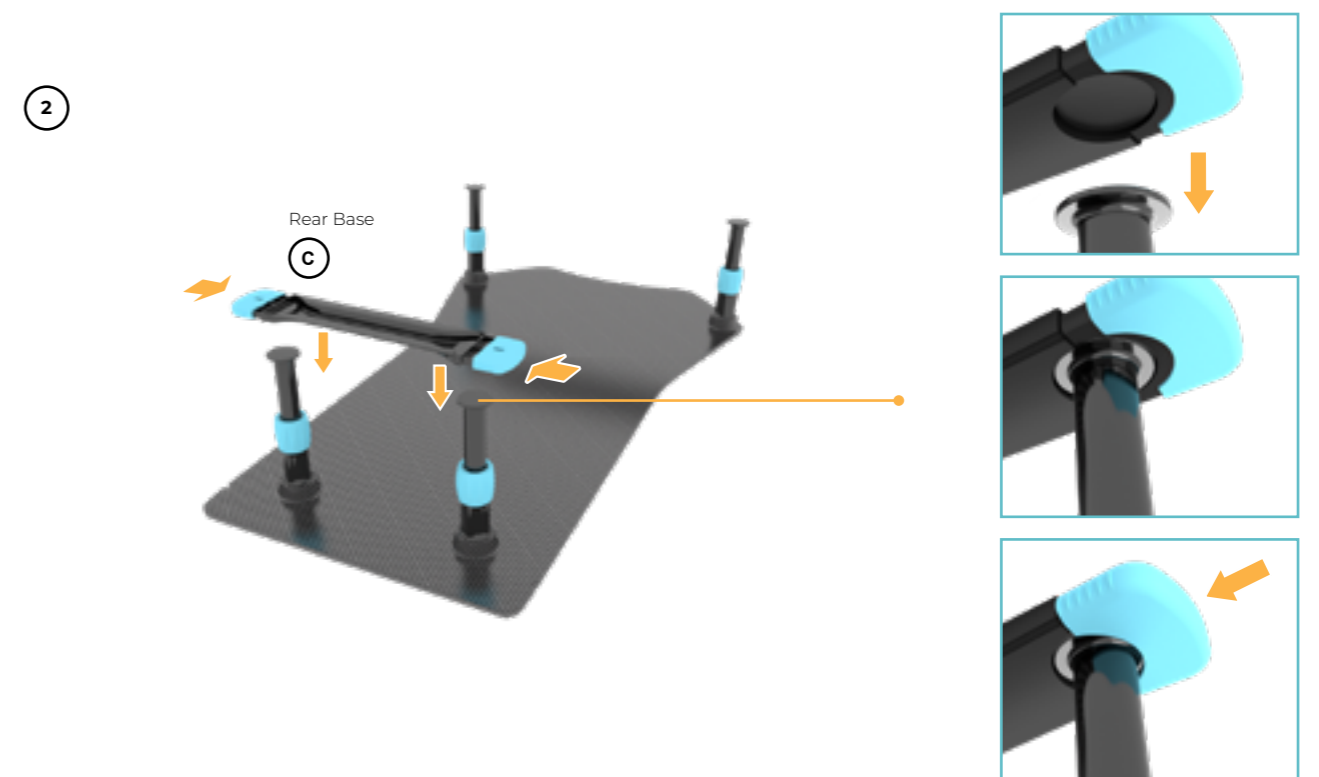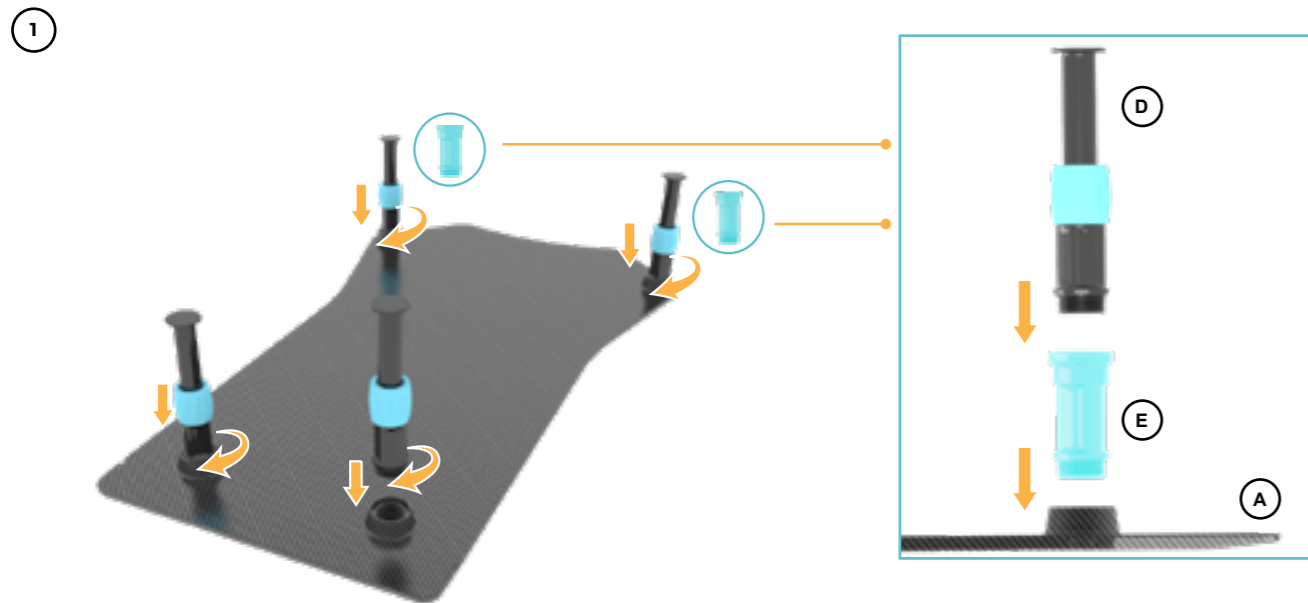

Setup

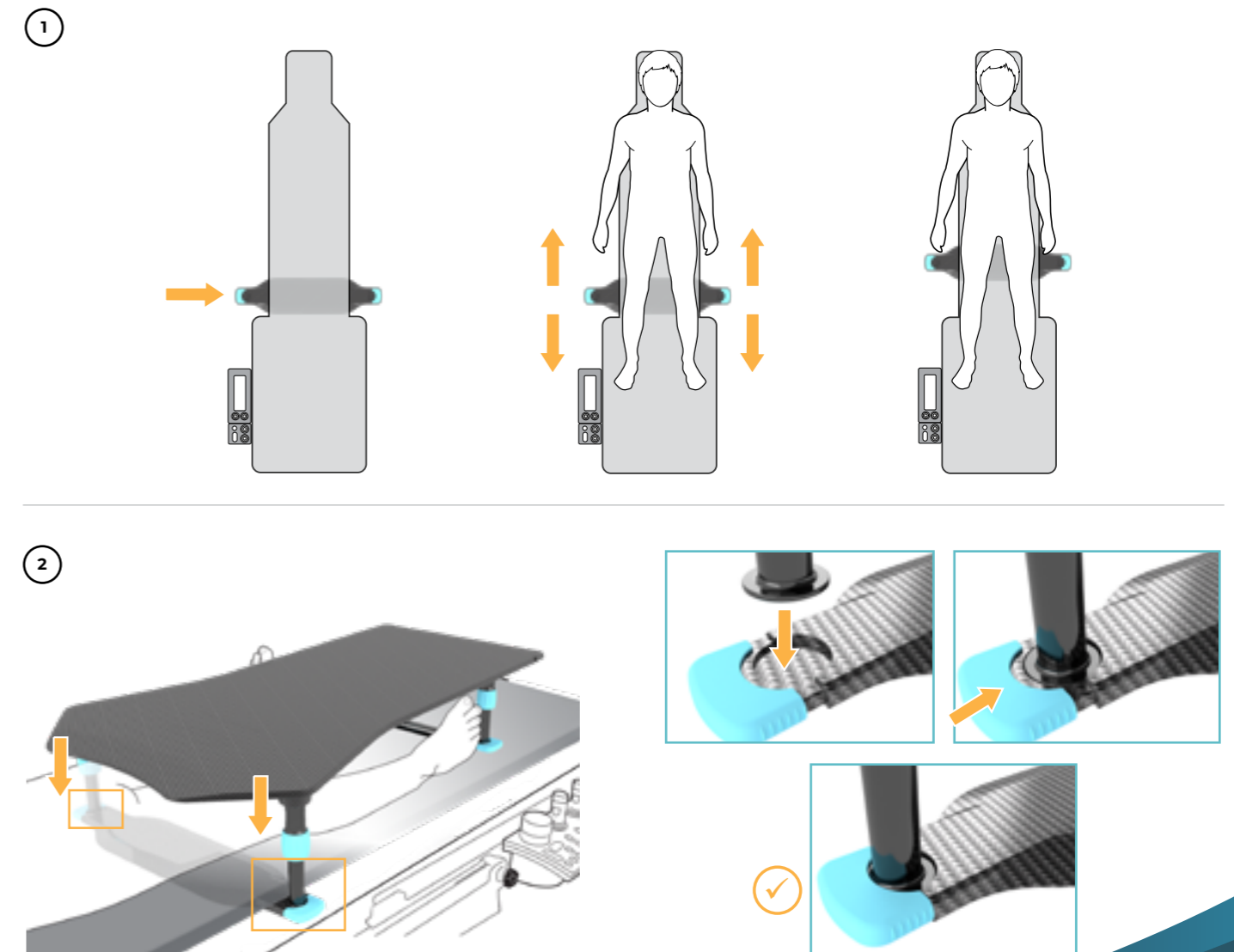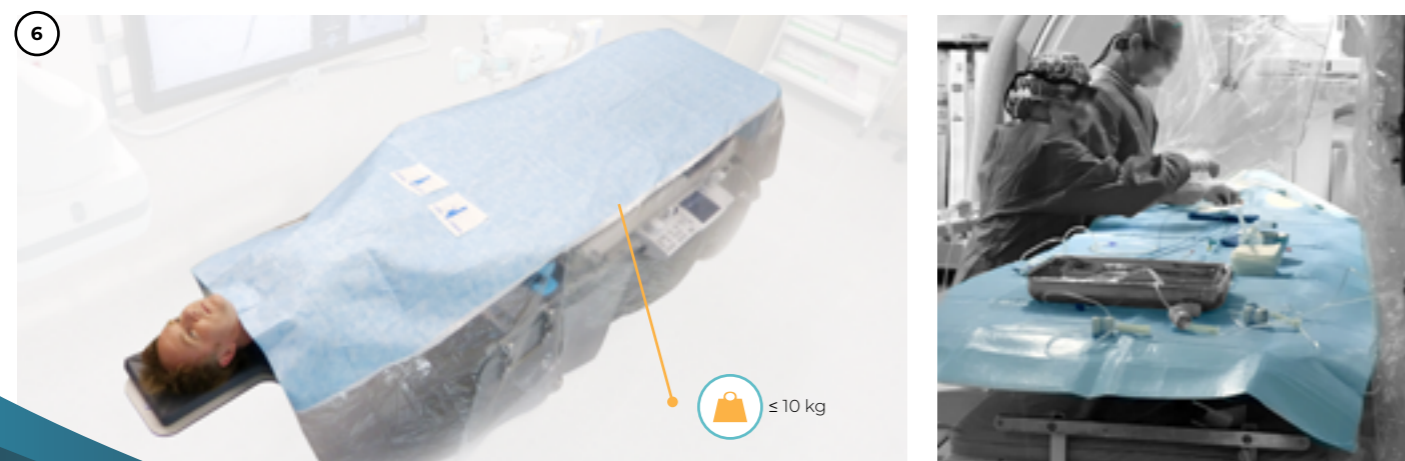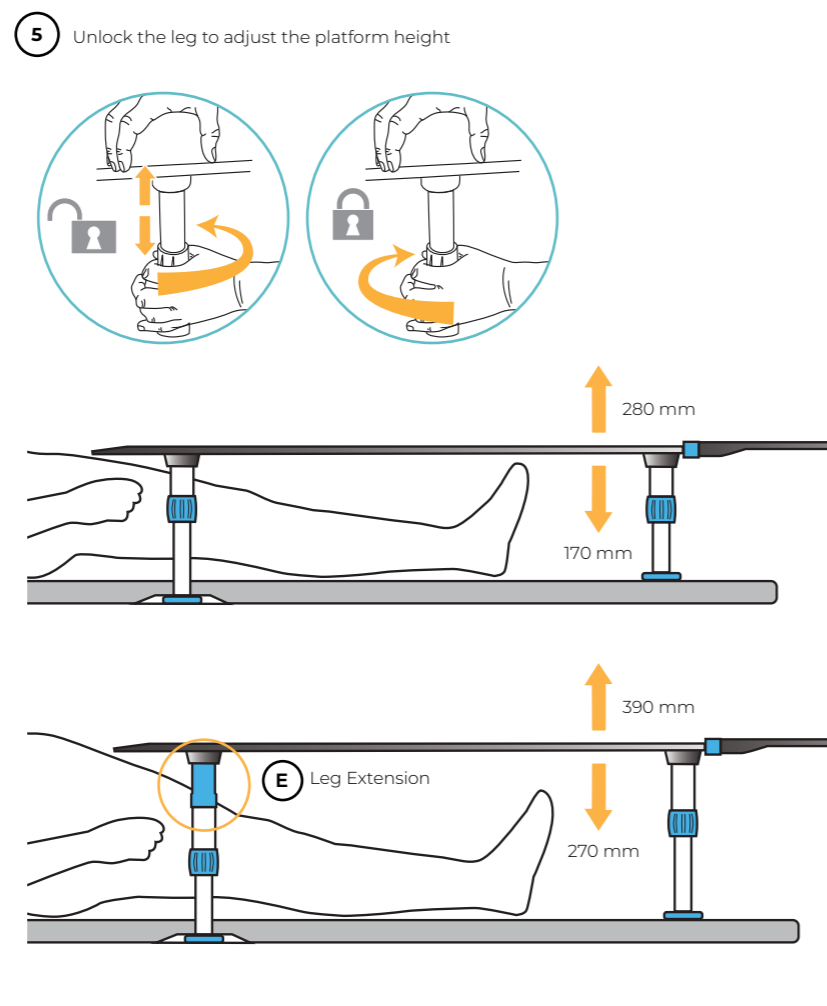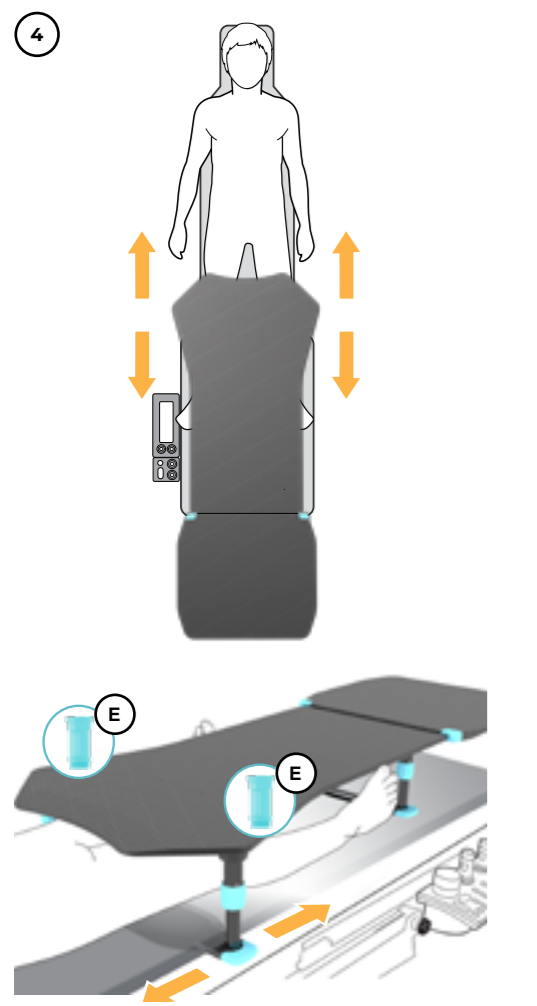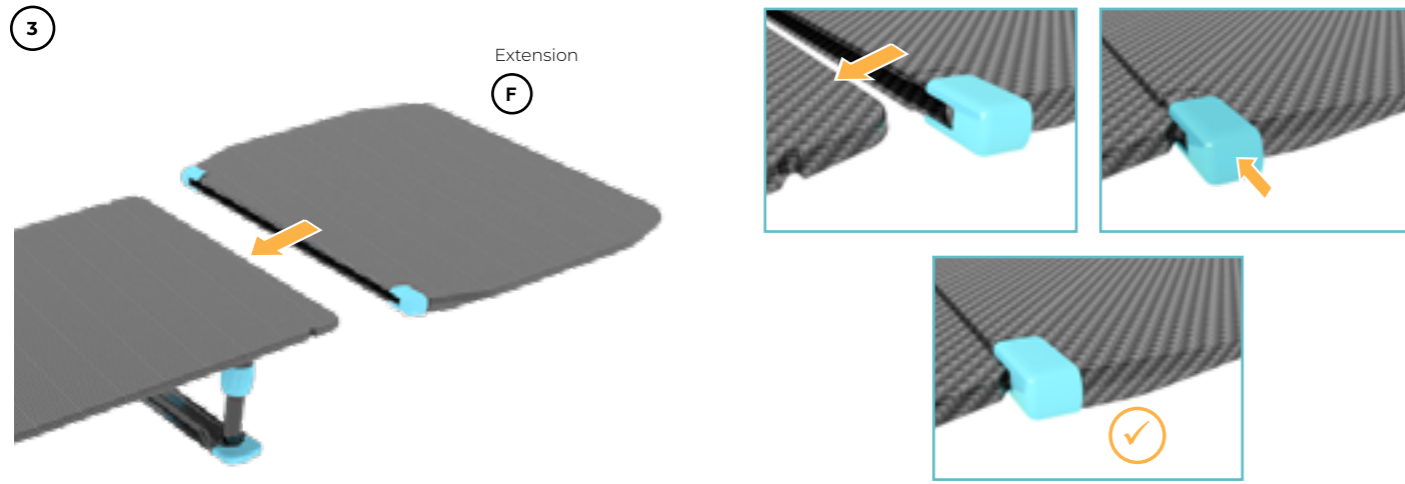Spare Parts

Product Code

	Retrograde IR Platform without Extension	AM0400
	IR Platform Extension	AM0410
	Retrograde IR Platform + Extension	AM0420

Component Codes

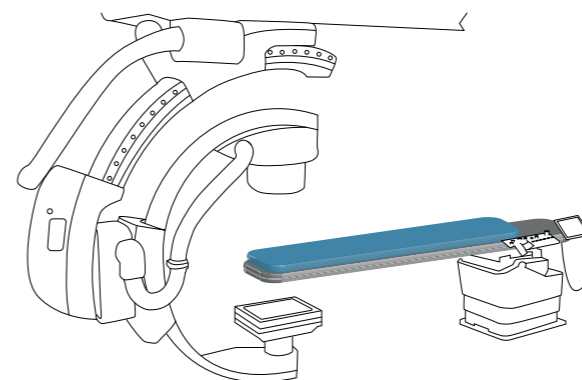
	A	Retrograde Main Platform	M3660
	B	Front Base	M3662
	C	Rear Base	M3663
	D	Leg	M3664
	E	Leg Extension	M3693
	F	Extension	AM0410

Image Table Verification

The product has been risk assessed to operate within the below specifications

Table Angulation: +/- 15° lateral & longitudinal
Table Surface: Flat

Essential Information

Information

Intended Purpose

To provide a work surface over a supine patient for femoral transarterial diagnosis and intervention via Retrograde approach on an Angiography imaging table.

Intended User/Training Requirement

Intended to be used by trained medical professionals.

Patient Target Group

Adult male or female patient requiring diagnostic imaging and image guided therapy.

Contraindications

- Not to be used with patients over 135 kg.
- Not for paediatric use.

Warnings and Cautions

- Ensure IFU is read prior to use.
- Ensure Preparatory Cleaning is conducted.
- Ensure Preparatory Product Check is conducted.
- Ensure all items placed on the Retrograde Main Platform do not exceed a total weight of 10 kg.
- Do not use with patients over 135 kg.
- Not for paediatric use.

Incident Reporting Guidelines

For product complaints and incidents, please complete form on www.adeptmedical.co.nz/repairs

Initial Checks & Cleaning

Cleaning

Refer to Disinfection Instructions.

Product Checks

- No exposed Carbon Fibre.
- No visible damage or sharp edges, e.g. cracks.
- Leg: Rotate all four blue locking collars to ensure they are free to rotate and smoothly lock and unlock.
- Leg: Ensure that the Leg is not free to move up and down in the locked position.
- Leg: Ensure that the Legs can move up and down when pushed in the unlocked position.
- Leg: Ensure the Legs are securely assembled on the Retrograde Main Platform (refer to page 4).
- Leg: Ensure Legs are secured in the Lower Bases (refer to page 5).
- Leg: Ensure Leg is locked during procedure (refer to page 6).
- Extension: Ensure the Extension is fully inserted on the Retrograde Main Platform and the Extension Clips are fully locked (refer to page 6).

Disinfection

Warnings

- Insufficient cleaning may compromise disinfection process and lead to transmission of pathogens.
- Do not use any disinfectant products or abrasive/corrosive agents which are not on the approved cleaners list.
- Always read manufacturer's instructions and consult the manufacturers MSDS for cleaning and disinfectant products.
- Do not use any cleaning procedure which exceeds 65 °C/149 °F.
- Some disinfectants may cause slight discolouration to the blue material used on some components within the product range. This will not affect the strength and the product will remain fit for purpose.

Limitations on Processing

Discontinue use if:

- Any cracks or breakages are present.
- Any exposed carbon fibre is visible.

Disinfection Instructions

Initial Treatment at the Point of Use

It is important to clean the product once it is removed from the packaging and after each procedure. Ensure all areas including joints, clips, sockets, brackets and levers are thoroughly cleaned after use to remove all contaminant build-up that may be present following a procedure.

Preparation Before Cleaning

- Remove the product from the table.
- Disassemble Retrograde IR Platform into Leg, Leg Extension, Rear Base, Front Base, Extension (where applicable) and Retrograde Main Platform.
- Ensure the Legs are fully extended.

www.adeptmedicaltraining.com/productcleaning/Retrograde-IR-Platform

Cleaning: Manual

- If any visible residue is present, rinse under water, but avoid submerging the device.
- A soft bristle brush can be used.
- Ensure the device is completely dried prior to disinfection.

Disinfection

Refer to the Approved Cleaners List for selection of an appropriate disinfecting agent.

Retrograde Main Platform:

- With a cloth dampened by an approved disinfectant, wipe all surfaces, including the threaded holes on the underside.

Extension (when applicable):

- With a cloth dampened by an approved disinfectant, wipe all surfaces, including entirety of slot and around the blue locking clips.

Front and Rear Base:

- With a cloth dampened by an approved disinfectant, wipe all surfaces, including leg sockets and blue locking clips.

Leg:

- With a cloth dampened by an approved disinfectant, wipe all surfaces.

Leg Extension:

- With a cloth dampened by an approved disinfectant, wipe all surfaces, including entirety of the thread.

Approved Cleaners List

Approved Disinfectant by Brands

Liquids:

ORION Laboratories - 70% Isopropyl Alcohol
Jaychem Industries - 2% Chlorhexidine / 70% Alcohol
Betadine - Povidone Iodine (7.5%)
BODE Chemie GmbH - Kohrsolin FF
STERIS - Coverage Spray TB
CaviCide - Metrex Research

Dissolvable:

Du Pont - Rely+On Virkon Tablets

Surface Wipes:

PDI Sani-Cloth Bleach
PDI Sani-Cloth AF3
Reynard Health Supplies - Surface Disinfectant Wipes
Metrex Research - CaviWipes
Clinell Wipes - Universal (Green)
Clorox Hydrogen Peroxide Wipes

Approved Active Disinfectant Ingredients

Acid Based:

≤10% Malic Acid CAS 6915-15-7
≤6% Sulfamic acid CAS 5329-14-6

Alcohol Based:

≤5% 2-Butoxyethanol CAS 111-76-2
≤10% Butyldiglycol CAS 112-34-5
≤70% Isopropyl Alcohol (Propan-2-ol) CAS 67-63-0
≤10% Tridecanol CAS 69011-36-5
≤10% Alcohols, C12-14, ethoxylated CAS 68439-50-9
≤70% Denatured Ethanol CAS 64-17-5

Sulphate Based:

≤0.1% PHMB CAS 27083-27-8
≤55% Potassium Peroxymonosulfate CAS 70693-62-8
≤3% Potassium Persulfate CAS 7727-21-1

Ammonium and Chloride Based:

≤5% Benzalkonium Chloride CAS 68424-85-1
≤0.28% Benzethonium Chloride CAS 121-54-0
≤10% Benzyl-C23-18-Alkyl-dimethyl Ammonium Chloride CAS 8001-54-5
≤2% Chlorhexidine CAS 55-56-1
≤10% Didecyl Dimethyl Ammonium chloride CAS 7173-51-5
≤0.5% Quaternary Ammonium Compounds CAS 68956-79-6
≤5% Benzyl-C 12-18 alkyl dimethyl ammonium chlorides CAS 63891-01-5

Others:

≤10% Glutaral CAS 111-30-8
≤0.63% Sodium Hypochlorite CAS 7681-52-9
≤7.5% Povidone Iodine CAS 25655-41-8
0.5% Disodium Cocoampho Dipropionate CAS68604-71-7
≤1.4% Hydrogen Peroxide CAS 7722-84-1

Access the latest Approved Cleaners list via www.adeptmedicaltraining.com/downloads

Drying

All components should be dried thoroughly before use.

Maintenance, Inspection and Testing

For all components ensure:

- No visible damage or sharp edges, e.g. cracks.

Retrograde Main Platform, Front Base, Extension (when applicable):

- No exposed Carbon Fibre is visible.

For the Extension (when applicable):

- Ensure the Extension can be assembled smoothly onto the Retrograde Main Platform and the locking clips are functional.

For the Front Base and Rear Base:

- Ensure the locking clips are functional and can retain the Leg.

For the Leg:

- Ensure thread is intact and can be screwed onto the Retrograde Main Platform.
- Ensure the blue collar rotates smoothly.
- Ensure when blue collar is in the unlocked position the Leg can extend and contract smoothly when force is applied.
- Ensure when blue collar is in the locked position the Leg cannot extend and contract when force is applied.

For the Leg Extension:

- Ensure thread is intact and can be screwed onto the Leg Extension and the Retrograde Main Platform.

Packaging

It is not required to package the device following disinfection.

Sterilisation

This device should not be subjected to sterilisation processes.

Storage

Once disinfection is complete and all components are dry the device should be stored in a dry environment.

Disposal

Used product is a biohazard, decontaminate according to instruction provided in the Disinfection Instructions and reach out to the manufacturer on adeptmedical@adept.co.nz for material specification, if required. Dispose according to internal clinic policy taking into consideration local regulations.

Serious Incidents

Any serious incident which occurs in relation to the device should be reported immediately to:

- The Competent Authority of the applicable Member State.
- The Manufacturer using the Serious Incident Reporting Form found on the Adept Medical Website: www.adeptmedical.com/form/serious-incident-reporting-form

The instructions provided above have been validated by the manufacturer of the medical device as being capable of preparing a medical device for reuse. It remains the responsibility of the processor to ensure that the processing, as actually performed using equipment, materials and personnel in the processing facility, achieves the desired result. This requires verification and/or validation and routine monitoring of the process.

Visit the Adept Medical Training Platform for detailed visual instructions related to this device. www.adeptmedicaltraining.com

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